paingone®*

the original TENS pen

**Fast,
Simple
Drug-free
Pain relief**

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Instructions for use Manuel d'instruction

*Paingone is a registered trademark. The name does not imply efficacy/performance. It is intended for temporary relief only, as part of your regular pain relief treatment plan.

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GB

How to use Paingone

Paingone is a low frequency TENS device powered by quartz crystals and designed to complement your current pain treatment plan. It is fully self-contained and does not require batteries. Paingone works using Transcutaneous Electrical Nerve Stimulation (TENS); a method for pain relief used widely around the world. Hold Paingone in your hand and firmly wrap your index finger around the metal ring. Place the tip of the device directly on the point you wish to use it and click the button 30-40 times. This should take about 60 seconds. Each click delivers TENS therapy straight to the point you need it. Re-apply when necessary, up to 4-5 times a day on each desired area. Paingone can even be used through light clothing. Paingone can also be used on acupoints to help enhance the therapy.

⚠ WARNING - DO NOT USE PAINGONE:

- If you use a pacemaker
- During the first 16 weeks of pregnancy
- If you suffer from epilepsy
- If you experience any irritation, itching or marking of the skin or any other adverse symptoms during or after application
- If the plastic body is broken in any way
- On the carotid sinus nerves, laryngeal or pharyngeal muscles in the front/side of your neck
- Over the heart or directly over metal implants
- If you suffer from heart rhythm problems
- While driving
- In and around your eyes
- On wet skin or in damp surroundings
- Close to oxygen or inflammable liquids
- On open wounds

Failure to follow these warnings could result in serious injury

⚠ CAUTIONS

- Read instructions before use
- Keep instructions in a safe place
- Not suitable for use by children
- Keep out of reach of children
- No product or advice is solely intended to replace current medication, diagnose or cure any conditions
- Please consult your doctor if you are concerned about the applicability of this product in relation to your own condition or if you are already taking steroids, warfarin or similar medication

Usage Instructions - Manuel d'instruction

Place the Paingone on the point of pain or acupuncture point and click the button 30 – 40 times, this should take around one minute.

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The electrical pulses of the TENS therapy may be felt with each click, though sensations vary by user

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Technical Information

Description: Hand-held, pen-like, piezoelectric TENS stimulator

Device Name: Paingone

Classification: Class IIa Medical Device (CE0086)

Applications: Paingone is designed for self-application and to deliver drug-free piezoelectric TENS therapy

Product Specification:

Approximate Dimensions: 32mm in length x 20mm in diameter

Mechanism of Action: Paingone is to be placed directly on the area you wish to use it or, alternatively, an acupuncture point. Pressing the button results in the generation of an electrical impulse which is delivered to the patient via the electrode touching the skin.

Energy Output: 2.6mJ at 2000Ω

Power Requirements: No power source is required

Operating Life: 24 months based on typical usage of three applications per day with 30-40 clicks per treatment.

Maintenance:

The device has been designed for maintenance-free use. Paingone does not require sterilization or disinfection. Paingone may be cleaned externally using a soft slightly damp cloth and allowed to dry before use.

Operating, Transportation & Storage Environment: Temperature: -20 °C to +50 °C Relative Humidity: 10% to 95%

Electromagnetic Compatibility:

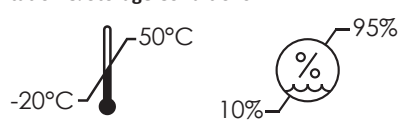
Paingone meets the standards required by the EN 60601 Electro Magnetic Compatibility (EMC) test (EN 60601-1-2:2007 and EN 61000-4-2:2008).

The operation of the Paingone is not affected by interference from other electrical or electronic equipment. Interference with electrical/electronic devices through the operation of the Paingone device has not been experienced but cannot be excluded.

Safety Standards: The Paingone conforms to EN 60601 - 1:1990 +A1:1993 +A2:1995

Quality Standards: The manufacturer meets the requirements of the Quality Assurance Standard (ISO 13485:2003). Paingone conforms to the European Medical Device Directive 93/42/EEC.

Transportation & Storage Conditions:

Symbols

- Manufacturer's Identification
- Attention, consult accompanying documents
- Protection against electric shock. Type BF applied part according to European Norm EN 60601-1.
- Degree of protection against the harmful ingress of water. Protected against vertically falling water drops.
- The battery should not be disposed of in normal household waste but returned via battery collection point.
- The equipment should not be disposed in the
- Identification of month & year of manufacture. LOT is given as yyyy/mm.
- CE marking of conformity to the European Medical Device Directive 93/42/EEC under the supervision of Notified Body Number 0086 British Standards Institution (BSI).

Medi-Direct International Ltd, Unit 24, Wilford, Business Park, Ruddington Lane, Nottingham, NG11 7EP, UK, www.paingone.com

© Copyright, 2017, Medi-Direct International Ltd. All rights reserved. Copying or other reproduction of this document is prohibited without the prior written consent of Medi-Direct International Ltd.

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12 month replacement warranty

Register your Paingone online today. In addition to the 12-month manufacturer’s warranty, you will benefit from a further 12-month replacement warranty in case of manufacturing fault.

If your Paingone develops a fault within 2 years of purchase, simply return the device with a copy of the purchase receipt. A replacement will be sent to you free of charge. Cover is from the date of receipt. A replacement cannot be given if we believe the malfunction is due to misuse.

Please visit www.paingone.com/warranty to register your extended Paingone warranty and for full terms and conditions and return address details.

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© Copyright, 2017, Medi-Direct International Ltd. Tous droits réservés. Toute reproduction totale ou partielle de ce document est interdite sans l'autorisation écrite préalable de Medi-Direct International Ltd.